ENGLISH

Please keep these instructions for future reference.

The Quick Rail is designed to assist your natural motion, as when getting in and out of a shower. **It is not designed to hold or support your full body weight.** The Quick Rail can be used where permanent drilling and fixing is not possible. Do not attach to moving surfaces.

INSTALLATION INSTRUCTIONS - Location:

The Quick Rail is designed for use only on smooth non-porous surfaces; tile, glass, porcelain, some plastics. Not for use on painted, papered, wood, embossed surfaces.

Check the condition of the support surface before positioning the Quick Rail.

Support surfaces should be sound and free from defects.

Do not mount the Quick Rail where it will be in direct sunlight.

Check the condition of the suction cups. Do not use if damaged, cracked or split.

Do not attempt to mount the Quick Rail on mosaic or embossed tiles.

Fitting

- Thoroughly clean the surface with a solvent cleaner or alcohol wipe and allow to dry.
- Raise both levers; place the Quick Rail so that both suction cups lie firmly and completely on a smooth surface.
- Do not place over joints or grout lines.
- Press on one lever at one end, then the other. You should feel resistance as the suction cup tightens.
- Check secure attachment before each use.

Removal

- Lift each of the two levers.
- If suction remains, gently lift one edge with a finger.
- Do not use a sharp instrument which may damage the suction cup.
- Clean the Quick Rail from time to time with a non-abrasive cleaner.

Liability Warning

The Quick Rail is designed for use only on smooth non-porous surfaces.

Check the stability of the mounting surfaces before installation.

Always check attachment before each use.

THE USER ASSUMES ALL RISK OF INJURY DUE TO USE

As the conditions in which the Quick Rail is used are beyond the control of the manufacturer or supplier, it is the obligation of the user to determine the correct and safe selection and conditions of use of the Quick Rail. To the extent that the law permits the supplier's liability is limited to the cost of repairing or replacing the products at its discretion, and does not extend to any loss or damage which may be caused as a consequence of misuse or improper or defective installation. In no event shall the supplier be liable for any other loss or damage, whether direct, indirect, foreseeable, consequential or special (including, but not limited to physical injuries or loss of use, revenue or profit) whether based on warranty, contract, tort (including negligence) or strict liability arising in connection with the use of the Quick Rail

The supplier makes no other warranty whatsoever, and specifically disclaims and excludes all other warranties of any kind or nature whatsoever, directly or indirectly, express or implied, including without limitation, as to the suitability, productivity, durability, fitness for a particular purpose or use, merchantability, condition, or any other matter with respect to the Quick Rail.

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